NAQI® HOT PACK

REUSABLE - WIEDERVERWENDBAR - REUTILISABLE - HERBRUIKBAAR

THERAPY FOR BACK AND MUSCLE PAIN AND OTHER NON-INFLAMMATORY RHEUMATIC DISORDERS

NAQI® Hot Pack, with three detachable sections for microwaves and warm baths. Very hygienic and time-saving solution for heat therapy applications. The NAQI® Hot Pack will stay warm for a minimum of 45 minutes.

INSTRUCTIONS

Microwave: Distribute the gel evenly along the NAQI® Hot Pack, especially in the corners. Heat for 3 minutes and then leave in the microwave to stand for a further 30 seconds. When the HotPack is not warm enough place another 30 seconds in the microwave. Apply the NAQI® HotPack with the protective sleeve. Remove from the microwave. Wait a further 30 seconds and apply the NAQI® Hot Pack. To re-apply the Hot Pack immediately, put it back in the microwave for a further 2 minutes. All timings are approximate and depend on microwave output. To start up using the NAQI® HotPack put the microwave on low wattage. (450w)

Warm water bath: Heat the water to 75°C. Leave the NAQI® Hot Pack to soak for 10 min.

Warning:

- Avoid contact with open wounds.
- Never heat the NAQI® Hot Pack beyond 70° C.
- Never bring into contact with sharp objects.
- Never place the pack on a heating element.
- Follow the instructions strictly.
- The NAQI® Hot Pack can be supplied with a customised non-woven Hot Pack sleeve 37 x 37 cm.

Measurement: 37 x 37 cm - model with three detachable sections.

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