NAQI® COLD/HOT PACK

REUSABLE COLD/HOT PACK FOR LONG-LASTING PAIN RELIEF

INSTRUCTIONS: Cold application & Hot application

Freezer: Place the NAQI® Cold/Hot Pack in the freezer. Leave it to freeze for at least 2 hours to obtain optimal results. Apply the frozen NAQI® Cold/Hot Pack to the injured area, which is covered by a dry towel or sleeve. NAQI® Cold/ Hot Pack stays flexible at subfreezing temperatures to conform to contours of the treated area. The pack is reusable. Return to the freezer after each use. Use only as directed. The intensity of the cold can be reduced by placing an additional dry towel or a sleeve between the skin and the NAQI® Cold/Hot Pack.

Microwave: Ensure the product is evenly distributed across the NAQI® Cold/Hot Pack. Heat the NAQI® Cold/Hot Pack for 1-2 minutes and the NAQI® Cold/Hot Pack Large for 4 minutes at low wattage (600/650 watt) and then leave in the microwave to stand for a further 30 seconds. Apply the NAQI® Cold/Hot Pack, which is covered by a dry towel or sleeve. To re-apply the NAQI® Cold/Hot Pack immediately, put it back in the microwave for a further 1 minute (NAQI® Cold/Hot Pack) or 2 minutes (NAQI® Cold/Hot Pack Large). Use only as directed. All timings are approximate and depend on microwave output. Every microwave works differently. Monitor the heating process meticulously during the first use of the NAQI® Cold/Hot Pack to prevent overheating.

Warning: Never put the NAQI® Cold/Hot Pack directly onto the skin. Avoid contact with open wounds.

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NAQI COLD HOT PACK	Everyday use	13cmx27cm
NAQI COLD HOT PACK LARGE	Knee, Shoulder, Back	27cmx37cm

NAQI® COLD PACK

REUSABLE - WIEDERVERWENDBAR - REUTILISABLE - HERBRUIKBAAR

INSTRUCTIONS: Place the NAQI® Cold Pack in the freezer section of refrigerator. Leave it to freeze for at least 2 hours to obtain optimal results. Apply the frozen NAQI® Cold Pack to the injured area, which is covered by a dry towel or sleeve. NAQI® Cold Pack stays flexible at subfreezing temperatures to conform to contours of the treated area. The pack is reusable. Return to the freezer after each use. Use only as directed. The intensity of the cold can be reduced by placing an additional dry towel or a sleeve between the skin and the NAQI® Cold Pack.

Warning: Never put the NAQI® Cold Pack directly onto the skin. Avoid contact with open wounds.

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In case of a serious incident that has occurred in relation to the NAQI® Cold/Hot Pack or NAQI® Cold Pack contact NAQI: tel. +32(0)13 460 190 - info@naqi.com - www.naqi.com

NAQI® nv/sa

Stadsbeemd 1037, B-3545 Halen, Belgium

UK RP: NAQI International Ltd

East Wing, Goffs Oak House, Goffs Lane, Goffs Oak, Hertfordshire, EN7 5GE UK

NAQI COLD PACK MINI	On face, after eye surgery	7cmx13cm
NAQI COLD PACK SMALL	Gynaecology, Ear, Nose, Throat	7cmx27cm
NAQI COLD PACK STANDARD	Everyday use	13cmx27cm
NAQI COLD PACK SPORT	Remains cold long	13cmx27cm
NAQI COLD PACK MEDIUM	Ankle, Elbow	25cmx27cm